

Parts List: Kit# N211 Onboard Battery

Item	Part #	Description	Qty
1	11017016	Onboard Bed Battery	1
2	900589-0	10-32 Phillips Head Screw	4
3	12671006	10-32 Hex Nylock Nut	4
4	11017056AL	Mounting Bracket	2
5	12160281	Cable Tie, 10"	5

Installation Instructions

Suggested Tools: #2 Phillips screwdriver, 3/8" hex wrench or nut driver

1. Raise the bed to a comfortable working height.
2. Remove the mattress.
3. Raise/incline the knee section to access the control box

⚠ Warning: Possible Injury. Disconnect the bed mains power cable from the wall outlet to avoid electrical shock.

Note: Bed is upside down in some of the following figures for visual clarity.

4. Using provided fasteners and brackets, attach the battery to the frame of the bed (Figure 1). The cable should be facing towards the foot of the bed and the edge of the battery should be about 5 inches from the split in the frame.
5. Route the cable along the same path as the back actuator cable as shown in Figure 2.
6. Lift the control box cable cover on the side furthest from the mains power cable, as shown in Figure 3.
7. Remove the plastic plug from the port labeled "DC".
8. Connect the battery to the control box by plugging the cable labeled "DC" into the port labeled "DC" (Figure 4). Reinstall the control box cable cover.
9. Secure the cable using cable ties. Create a small cable bundle using a cable tie in the corner of the frame, as shown in Figure 5.
10. Check for proper operation of the bed.

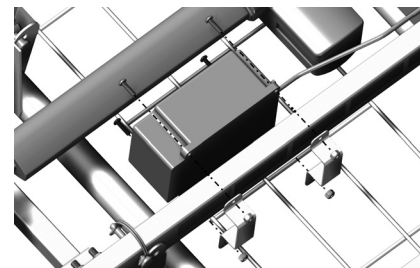


Figure 1

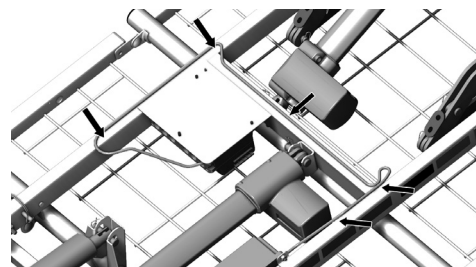


Figure 2

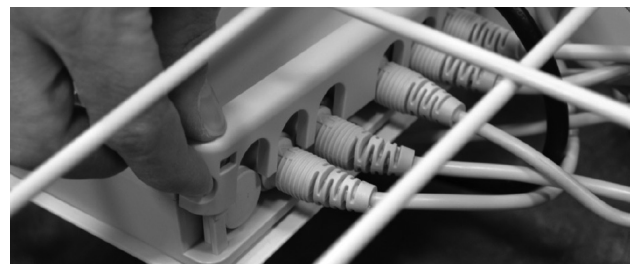


Figure 3

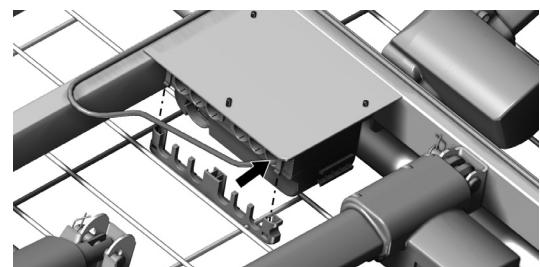


Figure 4

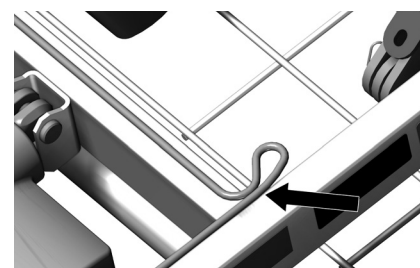


Figure 5

Customer feedback or questions about the enclosed bed accessory or service parts should be directed to your equipment provider.

Your equipment provider and contact information is noted on the bed frame Product Identification label or the bed User Manual.